



TelePointUSA

IVD
POCT



Telepoint SARS-CoV-2 IgM/IgM Rapid Test.
EUA authorized by the FDA

The **Telepoint SARS-CoV-2 IgG/IgM Rapid Test kit** is a qualitative test for the detection and differentiation of IgG and IgM antibodies against SARS-CoV-2 in human serum, plasma (potassium EDTA), and potassium EDTA venous whole blood. The product is intended for use as an aid in identifying individuals with an adaptive immune response to SARS-CoV-2, indicating recent or prior infection.

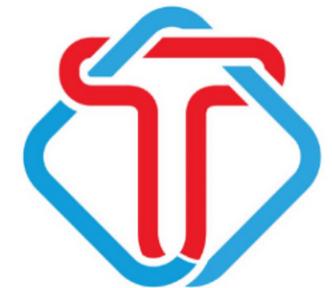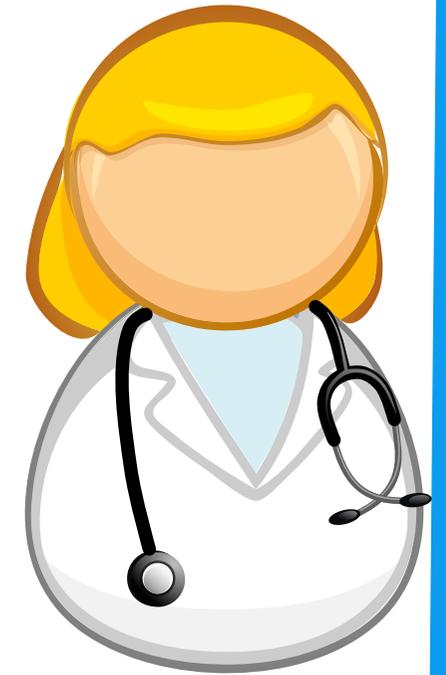


HOW IT WORKS:

The **Telepoint SARS-CoV-2 IgG/IgM Rapid Test kit** includes:

- (1) 25 Test Cassettes
- (2) 25 Droppers
- (3) 1 Bottle of Detection Buffer



READ BELOW

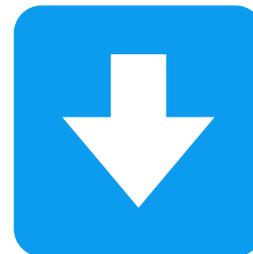

One drop of serum, plasma, or venous whole blood is transferred directly into the sample (S) well of the cassette followed by 3 drops of detection buffer into the D well. The mixture of specimen and SARS-CoV-2 antigen-coated colloidal gold particles migrates through the lateral flow strip to a region (M) where anti-human IgM antibody is immobilized and a region (G) where anti-human IgG antibody is immobilized. If IgM antibody against SARS-CoV-2 is present in the specimen, the particles are captured by the anti-human IgM causing the appearance of a colored line at the IgM test line region. Similarly, if IgG antibody against SARS-CoV-2 is present in the specimen, particles bind to anti-human IgG causing the appearance of a colored line at the IgG test line region.



If both IgM and IgG antibodies against SARS-CoV-2 are present in the specimen, colored lines appear at both the IgM and IgG test line regions. To serve as a procedural control, a colored line will always appear in the control line region (C), indicating that the proper volume of specimen has been added and membrane wicking has occurred. The appearance of a colored line only at the C region indicates that antibodies against SARS-CoV-2 were absent. If the control line fails to appear, due to insufficient volumes or incorrect procedural techniques, the test is invalid and must be repeated. All colored lines are read visually at 10 minutes and should be read no later than 10 - 20 minutes






TelePointUSA



Contact : LARRY WITHERSPOON

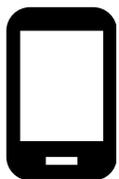

Mobile # (602) 529-4308



email to: Info@telepointusa.com Sales@telepointusa.com



Website : <http://telepointusa.com/>